


HitchFit® L-35 Positioning Chart

mm	-11.00	-8.80	-6.60	-4.40	-2.20	0.00	0.00	2.20	4.40	6.60	8.80	11.00	mm
11.00		E6	D6	C6	B6	A6	F1	F2	F3	F4	F5		11.00
8.80	F5	E5	D5	C5	B5	A5	E1	E2	E3	E4	E5	E6	8.80
6.60	F4	E4	D4	C4	B4	A4	D1	D2	D3	D4	D5	D6	6.60
4.40	F3	E3	D3	C3	B3	A3	C1	C2	C3	C4	C5	C6	4.40
2.20	F2	E2	D2	C2	B2	A2	B1	B2	B3	B4	B5	B6	2.20
0.00	F1	E1	D1	C1	B1	A1	A1	A2	A3	A4	A5	A6	0.00
0.00	A6	A5	A4	A3	A2	A1	A1	B1	C1	D1	E1	F1	0.00
-2.20	B6	B5	B4	B3	B2	B1	A2	B2	C2	D2	E2	F2	-2.20
-4.40	C6	C5	C4	C3	C2	C1	A3	B3	C3	D3	E3	F3	-4.40
-6.60	D6	D5	D4	D3	D2	D1	A4	B4	C4	D4	E4	F4	-6.60
-8.80	E6	E5	E4	E3	E2	E1	A5	B5	C5	D5	E5	F5	-8.80
-11.00		F5	F4	F3	F2	F1	A6	B6	C6	D6	E6		-11.00
mm	-11.00	-8.80	-6.60	-4.40	-2.20	0.00	0.00	2.20	4.40	6.60	8.80	11.00	mm

This positioning chart shows the 117 distinct adjustment positions available with the L-35 prosthetic alignment device. The reference view is from the foot toward the socket, with the toes positioned at the front and the heel at the back. Use this chart in conjunction with the positioning illustration on the next page to determine the correct product orientation and adjustment locations.

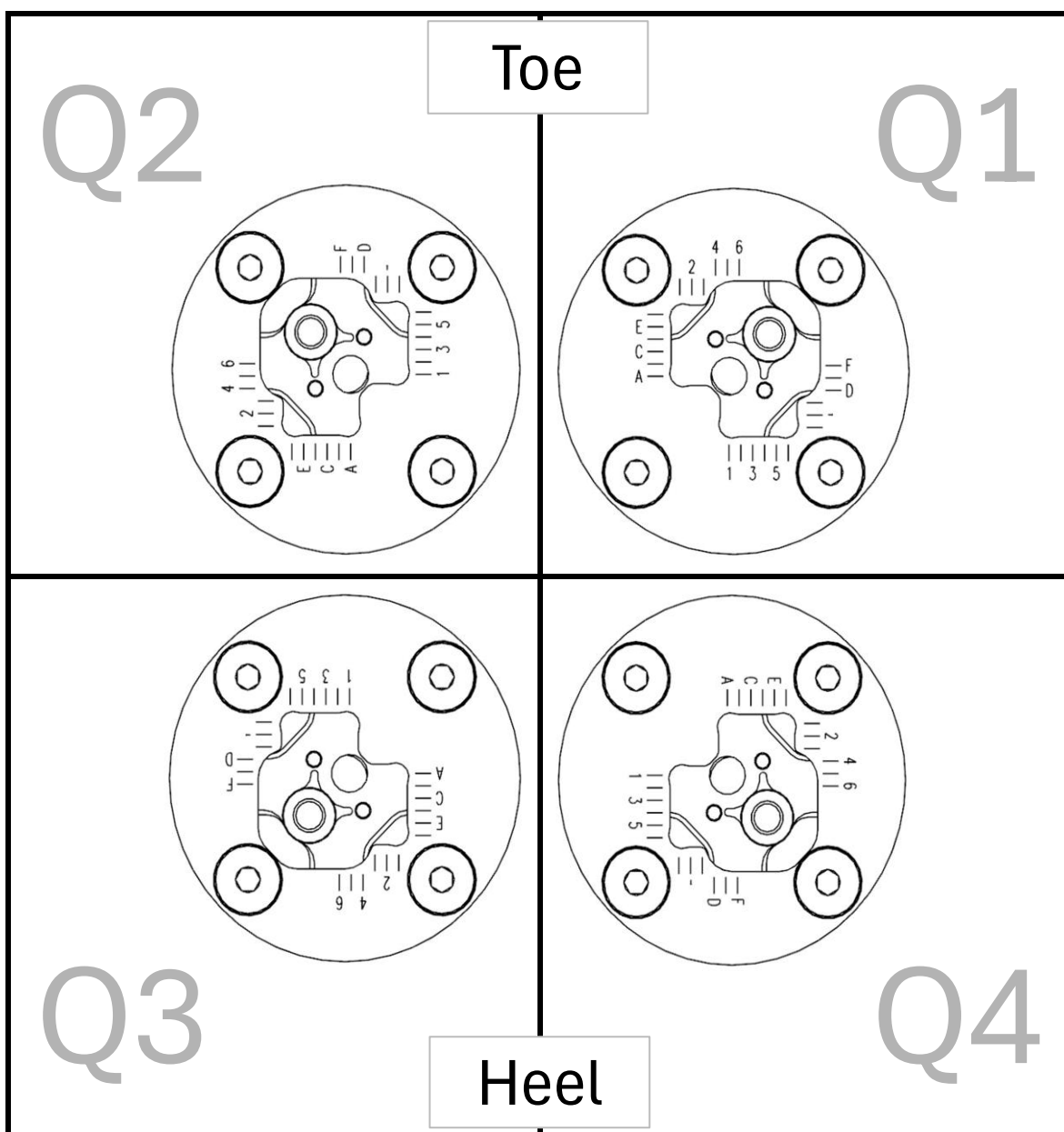

This positioning illustration defines the four quadrant orientations used for device alignment. The reference view is from the foot toward the socket, with the toes positioned at the front and the heel at the back. To achieve a Quadrant 1 configuration, install the device as shown in the “Q1” section of the illustration, maintaining the reference view perspective.